

Prosthetic Feet

DuraLite™ Foot

Patients who need a very lightweight foot with high-energy return and prosthetists who want a foot that is easily field-height adjustable now have the ideal choice with the DuraLite™ Foot. The foot incorporates a urethane toe pad which enhances medial and lateral motion. The DuraLite's split heel conforms to the floor and provides stability on uneven terrain. The choice of a pyramid adapter or pyramid receiver is offered for a proximal adapter. A unique feature of this foot is a Stair Guard. Mounted immediately beneath the proximal adapter, the Stair Guard helps to reduce the risk of the shin adapter catching on a step when descending stairs.

The DuraLite foot provides active amputees a light-weight, responsive foot for their prostheses. Amputees who tested the DuraLite foot spoke highly of its light weight and responsiveness in daily activities which included yard work, hiking and jogging.



- **Moderate to High Activity (U.S. Levels 3)**
- **Adjust Shank to Suit Patient Needs**
- **Exceptionally Lightweight**
- **300 lb Weight Limit**

SPECIFICATIONS

INCLUDES	High-Activity Foot Shell, Spectra Sock, Pyramid Receiver or Rotating Pyramid, 1 of 6 Toe Spring Options, 3 Heel Wedges
WEIGHT	356 g*
ADAPTER WEIGHTS	Pyramid 205 g; Receiver 190 g
PATIENT WEIGHT LIMIT	Foot Size 23-24 cm: 230 lbs (104 kg)** Foot Size 25-26 cm: 270 lbs (122 kg)** Foot Size: 27-31 cm: 300 lbs (135 kg)**
HEEL HEIGHT	3/8" (10 mm) or 3/4" (18 mm)
CLEARANCE***	13" (330 mm) maximum
Pyramid Receiver:	6.0" (152 mm) minimum for 3/4" heel 6.4" (162 mm) minimum for 3/8" heel
Pyramid Adapter:	6.15" (156 mm) minimum for 3/4" heel 6.55" (166 mm) minimum for 3/8" heel
AVAILABLE SIZES	23 to 31 cm
TOE RESISTANCE	Six - See Composite Toe Spring Chart
HEEL SPRING OPTIONS	Three Wedges (included with each foot)
COSMESIS	Lifelike in Buff, Tan or Medium Brown
TRIAL PERIOD	60 days from invoice date
U.S. WARRANTY	36 months from purchase
U.S. L-CODE SUGGESTION	5980

* Based on size 26 cm foot, without Proximal Adapter, Foot Shell, or Spectra Sock.

** Body weight plus any loads normally or routinely carried cannot exceed this weight limit.

*** Height calculated using Magnum Titanium 4-Hole Pyramid Receiver (MHD-PR-4) with pyramid adapter and Magnum Titanium 4-Hole Pyramid (MHD-PA-4) with receiver.



COSMESIS

Medium Brown, Tan or Buff



Prosthetic Feet

ORDERING GUIDELINES

To order a DuraLite Foot, please select the size, side, color, toe spring stiffness, and heel height.

	DLF-23LB-4L-1	DuraLite Foot SAMPLE Part Number
Name Code	DLF	DLF = DuraLite Foot
Size of Foot	23	23 to 31 cm
Side of Foot	L	L = Left R = Right
Foot Shell Color	B	B = Buff M = Medium Brown (special order) T = Tan (special order)
Toe Spring Stiffness	4	1 to 6 (refer to chart below)
Heel Height	L	L = Low 10 mm (3/8") M = Medium 18 mm (3/4")
Proximal Adapter	1	1 = Pyramid Adapter 2 = Pyramid Receiver

GENERAL GUIDELINES

To Select Proper Composite Toe Spring Stiffness:

1. Calculate adjusted body weight. (Adjusted body weight = patient body weight + loads **normally or routinely** carried)
2. Using adjusted body weight and patient activity level, select the toe spring stiffness from the chart below.

COMPOSITE TOE SPRING STIFFNESS CHART

		Foot Size		
		23 -24 cm	25-26 cm	27-31 cm
100-120 lbs (45-54 kg)	Medium Activity	Category 1	Category 1	Category 1
	High Activity	Category 2	Category 2	Category 2
121-150 lbs (55-68 kg)	Low Activity	Category 1	Category 1	Category 1
	Medium Activity	Category 2	Category 2	Category 2
	High Activity	Category 3	Category 3	Category 3
151-190 lbs (69-86 kg)	Low Activity	Category 2	Category 2	Category 2
	Medium Activity	Category 3	Category 3	Category 3
	High Activity	Category 4	Category 4	Category 4
191-230 lbs (87-104 kg)	Low Activity	Category 3	Category 3	Category 3
	Medium Activity	Category 4	Category 4	Category 4
	High Activity	Category 5	Category 5	Category 5
231-270 lbs (105-122 kg)	Low Activity	—	Category 4	Category 4
	Medium Activity	—	Category 5	Category 5
	High Activity	—	—	Category 6
271-300 lbs (123-135 kg)	Low Activity	—	—	Category 5
	Medium Activity	—	—	Category 6