

Prosthetic Feet

Carbon Copy® High Performance Foot

The Performance Solution

The Carbon Copy High Performance Foot is designed for highly active amputees. Its unique design responds to both low and high activity levels, making this the foot of choice for active amputees who enjoy less active activities as well. An inventive keel/plate attachment allows energy storage in mid-stance. In addition, the molded-in titanium pyramid reduces the foot's overall weight. The Carbon Copy High Performance offers a one-year warranty, and carries a weight load up to 250 lbs (115 kg).



- Moderate to High Activity (U.S. Levels 3 & 4)
- Keel/Plate Attachment Allows Energy Storage at Mid Stance
- Molded-in Titanium Pyramid Reduces Weight
- Available in Three Heel Heights
- All Three Heel Inserts are Included with Each Foot

SPECIFICATIONS

WEIGHT	Mens: 633 g (1.40 lb)* Womens: 587 g (1.29 lb)*
PATIENT WEIGHT LIMIT	250 lb (115 kg)**
HEEL HEIGHT	Women - 3/8" (10 mm) or 1-1/8" (30 mm) Men - 3/4" (18 mm) or 3/8" (10 mm)
AVAILABLE SIZES	Women - 22 to 27 cm Men - 23 to 31 cm
TOE RESISTANCE	Very Low, Low, Regular High, Very High
HEEL INSERT DENSITIES	Soft, Medium, Firm (All three are included with each foot)
COSMESIS	Lifelike in Buff, Tan or Medium Brown
U.S. WARRANTY	One year and three weeks from purchase
U.S. L-CODE SUGGESTION	5981

* Based on size 26 cm foot.

** Body weight plus any loads normally or routinely carried cannot exceed this weight limit.



COSMESIS
Medium Brown, Tan or Buff



Prosthetic Feet

ORDERING GUIDELINES

To order a Carbon Copy HP Foot, please select the gender, size, side, color, toe resistance and heel height. All three heel inserts are included with every foot.

	CHPW-22LB-4M	Carbon Copy HP SAMPLE Part Number
Name Code	CHPW	CHPM = Carbon Copy HP - Mens Foot CHPW = Carbon Copy HP - Womens Foot
Size of Foot	22	Mens = 23 to 31 cm Womens = 22 to 27 cm
Side of Foot	L	L = Left R = Right
Color	B	B = Buff M = Medium Brown T = Tan
Toe Resistance	4	1 = Very Low 2 = Low 3 = Regular 4 = High 5 = Very High
Heel Height	M	L = Low 10 mm (3/8") M = Medium 18 mm (3/4") Mens only H = High 30 mm (1-1/8") Womens only

GENERAL GUIDELINES

The patient's body weight plus any loads normally or routinely carried must not exceed 250 lbs (115 kg). This "modified body weight" is used to determine toe resistance. If the "modified body weight" of the patient exceeds 250 lbs (115 kg), the Pathfinder® II Foot, Magnum Foot or Magnum System is recommended.

Use the patient's foot size and "modified body weight" to determine the recommended toe resistance from the chart below. If the patient is moderate to high activity (U.S. Level 3 or 4), increase the toe resistance to the next higher resistance. If you choose a toe resistance lower than what our guidelines indicate, ensure that the patient's "modified body weight" does not exceed the maximum weight indicated in the chart by more than 20 lbs.

TOE RESISTANCE

Foot Size	Very Low (1)	Low (2)	Regular (3)	High (4)	Very High (5)
Womens 22-23 cm	70 lbs and below	71-115 lbs	116-160 lbs	161-195 lbs	196-250 lbs
	32 kg and below	33-52 kg	53-73 kg	74-88 kg	89-115 kg
Womens 24-25 cm	80 lbs and below	81-125 lbs	126-165 lbs	166-200 lbs	201-250 lbs
	36 kg and below	37-57 kg	58-75 kg	76-90 kg	91-115 kg
Womens 26-27 cm	90 lbs and below	91-130 lbs	131-170 lbs	171-205 lbs	206-250 lbs
	41 kg and below	42-59 kg	60-77 kg	78-93 kg	94-115 kg
Mens 23-24 cm	80 lbs and below	81-125 lbs	126-165 lbs	166-200 lbs	201-250 lbs
	36 kg and below	37-57 kg	58-75 kg	76-90 kg	91-115 kg
Mens 25-26 cm	90 lbs and below	91-130 lbs	131-170 lbs	171-205 lbs	206-250 lbs
	41 kg and below	42-59 kg	60-77 kg	78-93 kg	94-115 kg
Mens 27-28 cm	100 lbs and below	101-135 lbs	136-175 lbs	176-210 lbs	211-250 lbs
	45 kg and below	46-61 kg	62-79 kg	80-95 kg	96-115 kg
Mens 29-31 cm	110 lbs and below	111-145 lbs	146-180 lbs	181-215 lbs	216-250 lbs
	50 kg and below	51-66 kg	67-82 kg	83-98 kg	99-115 kg